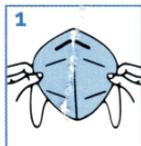
FFP2 Atemschutzmasken

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10 Stück

FFP2 NR – Model 4270001575944

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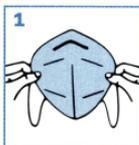
FFP2 NR – Model 4270001575944

USER INSTRUCTIONS

1. Place the mask in your hands with the top side facing upwards.

2. Put on the protective mask and press the metal part on the bridge of your nose, so that it adapts the shape of your nose.

3. Pull the bottom of the mask slightly down so that nose, mouth and chin are well covered.

4. Check the fit of the mask so that no secondary air is drawn in. Check facial hair for air leaks. If necessary, correct the fit until the mask fits comfortable and seals correctly.

The breathing masks are for single use.

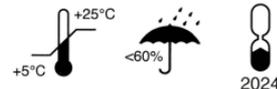
Please dispense after use. Please dispense the mask in event of moisture, visible contamination or inadequate fit. Replace with a new, dry, clean and well-fitting FY20-05 mask. Shelf life of the product: 3 years when stored at temperatures from 5 ° C to + 25 ° C and at a maximum Humidity of 60%. The face mask complies with DIN EN 149: 2009-08 and PPE Personal Protective Equipment (PPE) – Regulation (EU) 2016/425. The NR marking means that the mask may be used for a maximum of one working shift (8 hours). Disregard of the instructions or incorrect application may reduce the effectiveness of the mask or result in illness or permanent damage.

EU declaration of conformity:

<https://ppegermany.de/eu.html>

Production: January 18, 2021

Batch number: 20120901

Produced by:

PPE Germany GmbH
Karlsruher Str. 18
D-10711 Berlin

Contact:

maske@ppegermany.de

Item number:

4270001576156

CE
2163